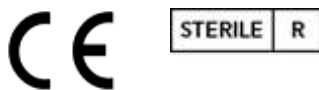
#4219 EMTRAC Plus

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#4219 EMTRAC Plus
Embryo Transfer Catheter
#4219 Emtrac Plus

Important product information
Read carefully before use

Disposable – discard after single use
Re-use may cause sexually transmitted infections
Caution: federal (USA) law restricts this device to be used by a physician
Sterile: unless pouch is damaged or opened

Indications for Use:

Introduction of embryos into the uterine cavity.

Contraindications:

Do not use in the presence of cervical infection.
Do not use in the presence or after recent Pelvic Inflammatory disease.
Do not use for intrafallopian procedures.

Warnings:

Not sterile if pouch is damaged or opened, discard immediately.

Precautions:

By virtue of its flexibility and small diameter, the Emtrac Plus can be introduced into the uterus with minimal danger of uterine wall perforation. However, care should be taken, prior to its insertion, to ascertain the depth of the uterus and any uterine antelexion or retroflexion present in individual patients. (This is preferably done during a previous examination and not on the day of the artificial insemination when disturbance of the uterus should be minimized).

In no case should the device be forced against digitally felt resistance.

If passage of the Emtrac - Plus through the cervical canal is not found to be easily achievable, the device should not be forced and the possibility of the presence of pathologic cervical stenosis considered.

The Emtrac Plus should be used by healthcare professionals only.

Adverse reactions:

No adverse reactions associated with the use of the Emtrac #4219 Plus have been reported. However, as with any instrument, which passes through the internal cervical os, mild cramping may be expected. In every case the direction for use should be followed; taking note of contra-indications, warnings and precautions.

Manufactured by:
Gynetics Medical Products N.V.
Rembert Dodoensstraat 51
3920 Lommel, Belgium

How supplied:
#4219 Emtrac Plus.

Packaged:
3 boxes of 25 individually sterile packed, disposable devices.

Directions for use:

1. With the patient in the dorsal lithotomy position expose the uterine cervix to view with a vaginal speculum.
2. Set the positioner as previously measured.
3. Having previously determined the depth and direction of the patient's uterus, gently advance the Emtrac A (white handle) with the Emtrac B (blue handle) catheter into the external cervical os and advance it through the cervical canal into the uterine cavity until resistance is felt. The distal tip of the catheter should then be at the uterine fundus. Now retract the catheter approximately 1.0 cm by following the guides. Gently rotate the catheter to remove any excess mucus from the tip.
4. Remove the Emtrac B (blue handle) when the position is reached.

Note: If any difficulty with insertion or bending of the catheter occurs, gently rotate the catheter while advancing it close to the cervix. If the uterus is significantly flexed anteriorly or posteriorly, gentle traction on the cervix with a tenaculum may be required to accomplish full insertion. In no case should force be used against resistance to achieve insertion.

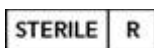
5. Load the embryo(s) into the Emtrac A (white handle) following your standard protocol.
6. With the cannula in position, release the embryos.
7. Following the complete expulsion of the embryos, slowly withdraw the catheter from the uterus and discard it. Remove the speculum and allow the patient to rest.
8. Once the catheter is removed from the patient, inspect the catheter to verify embryo release.

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