#4000 The Tulip

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#4000 The Tulip A

0120

Important information, read carefully before use

Disposable – discard after single use**Re-use may cause sexually transmitted infections****Caution: federal (USA) law restricts this device to sale by or on the order of a physician****Sterile: unless pouch is damaged or opened****Indications for Use:**

Introduction of embryos into the uterine cavity.

Contraindications:

Do not use in the presence of cervical infection.

Do not use in the presence or after recent Pelvic Inflammatory disease.

Do not use for intrafallopian procedures.

**Warnings:**

Not sterile if pouch is damaged or opened, discard immediately.

Precautions:

By virtue of its flexibility and small diameter, #4000 The Tulip / #4000 The Tulip Memo can be introduced into the uterus with minimal danger of uterine wall perforation. However, care should be taken, prior to its insertion, to ascertain the depth of the uterus and any uterine anteversion or retroversion present in individual patients. (This is preferably done during a previous examination and not on the day of the artificial insemination when disturbance of the uterus should be minimized).

In no case should the device be forced against digital resistance.

If passage of #4000 The Tulip / #4000 The Tulip Memo through the cervical canal is not found to be easily achievable, the device should not be forced and the possibility of the presence of pathologic cervical stenosis considered.

The Tulip should be used by healthcare professionals only

Adverse reactions:

No adverse reactions associated with the use of #4000 The Tulip / #4000 The Tulip Memo have been reported. However, as with any instrument, which passes through the internal cervical os, mild cramping may be expected. In every case the direction for use should be followed; taking note of contra-indications, warnings and precautions.

**Manufactured by:**

Gynetics Medical Products N.V.
Rembert Dodoensstraat 51
3920 Lommel, Belgium

How supplied:

Product number: - #4000 The Tulip
- #4000 The Tulip Memo
- #4000 The Tulip Memo B
- #4000 The Tulip A

Packaged:

- #4000 The Tulip / #4000 Tulip Memo: 3 boxes of 25 individually sterile packed, disposable sets of devices.
- #4000 The Tulip Memo B: 5 mandrels per 25 Tulip Memo.
- #4000 The Tulip A: 3 boxes of 25 individually sterile packed guiding catheters.

Directions for use:

1. With the patient in the dorsal lithotomy position expose the uterine cervix to view with a vaginal speculum.
 2. Having previously determined the depth, set positioner on the guiding catheter, #4000 The Tulip A / #4000 The Tulip Memo A (white handle).
 3. Having previously determined direction of the patient's uterus, put the mandrel, #4000 The Tulip B / #4000 The Tulip Memo B (blue handle), in guiding catheter #4000 The Tulip A / #4000 The Tulip Memo (white handle) to pre-shape The Tulip A / The Tulip Memo A.
 4. Gently advance the catheter into the external cervical os and advance it through the cervical canal near the uterine cavity until resistance is felt.
 5. Remove the mandrel #4000 The Tulip B / #4000 The Tulip Memo B (blue handle) when position is reached
 6. Load the embryo(s) into the loading catheter, #4000 The Tulip C / #4000 The Tulip Memo C (pink handle) following your standard protocol. Insert the loading catheter. The marking on the proximal end of the tube indicates when the distal tip of the guiding catheter is reached.
 7. The distal tip of the loading catheter should then be at the uterine fundus.
- Note:** If any difficulty with insertion or bending of the catheter occurs, gently rotate the catheter while advancing it close to the cervix. If the uterus is significantly flexed anteriorly or posteriorly, gentle traction on the cervix with a tenaculum may be required to accomplish full insertion. In no case should force be used against resistance to achieve insertion.
8. Slowly release the embryos into the uterus.
 9. Following the complete expulsion of the embryos, slowly withdraw the catheter from the uterus and discard it. Remove the speculum and allow the patient to rest.
 10. Once the set is removed from the patient, inspect The Tulip C / #4000 The Tulip Memo C to verify embryo release.
 11. Discard after use

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