#2000 SEMTRAC W

Soft Embryo Transfer Catheter
 #2000 Semtrac W-240 / #2000 Semtrac W-240 SP
 #2000 Semtrac W-180 / #2000 Semtrac W-180 SP
 #2000 Semtrac W-180 T / #2000 Semtrac W-240 T
 #2000 B W-180 / #2000 B W-240

Important information, read carefully before use

Disposable – discard after single use
Re-use may cause sexually transmitted infections
Caution: federal (USA) law restricts this device to sale by or on the order of a physician.

These devices are intended to support the use of an approved embryo transfer catheter. These tools are not intended to directly transfer gametes/embryos into the uterine cavity; rather, they are to aid in traversing a difficult or stenotic cervix or to perform a trial/mock transfer to better identify the uterine environment.

Sterile: unless pouch is damaged or opened

Indications for Use:

Introduction of embryos into the uterine cavity.

Contraindications:

Do not use in the presence of cervical infection.
 Do not use in the presence or after recent Pelvic Inflammatory disease.
 Do not use for intrafallopian procedures.

**Warnings:**

Not sterile if pouch is damaged or opened, discard immediately.

Precautions:

By virtue of its flexibility and small diameter, the #2000 Semtrac Set W can be introduced into the uterus with minimal danger of uterine wall perforation. However, care should be taken, prior to its insertion, to ascertain the depth of the uterus and any uterine antelexion or retroflexion present in individual patients. (This is preferably done during a previous examination and not on the day of the artificial insemination when disturbance of the uterus should be minimized).

In no case should the device be forced against digitally felt resistance.

If passage of the #2000 Semtrac W through the cervical canal is not found to be easily achievable, the device should not be forced and the possibility of the presence of pathologic cervical stenosis considered.

The Semtrac W should be used by healthcare professionals only

Adverse reactions:

No adverse reactions associated with the use of the #2000 Semtrac W have been reported. However, as with any instrument, which passes through the internal cervical os, mild cramping may be expected. In every case the direction for use should be followed; taking note of contra-indications, warnings and precautions.

Manufactured by:

Gynetics Medical Products N.V.
 Rembert Dodoensstraat 51
 3920 Lommel, Belgium

How supplied:

Product number:

- #2000 Semtrac W-240/ #2000 Semtrac W-180
- #2000 Semtrac W-240 SP/ #2000 Semtrac W-180 SP
- #2000 Semtrac W-180 T/ #2000 Semtrac W-240 T
- #2000 B W-180/ #2000 B W-240

Packaged:

- #2000 Semtrac W-240/ #2000 Semtrac W-180/ #2000 Semtrac W-180 T/ #2000 Semtrac W-240 T : 3 boxes of 25 individually sterile packed, disposable sets of devices and 5 mandrels.
- #2000 Semtrac W-240 SP/ #2000 Semtrac W-180 SP: 3 boxes of 25 individually sterile packed, disposable sets of devices and 5 mandrels.

Directions for use for the #2000 Semtrac W-240 / #2000 Semtrac W-240 SP / #2000 Semtrac W-180 / #2000 Semtrac W-180 SP:

1. With the patient in the dorsal lithotomy position expose the uterine cervix to view with a vaginal speculum.
 2. Having previously determined the depth and direction of the patient's uterus, gently advance guiding catheter #2000 Semtrac W - A (white handle) cervix cannula into the external cervical os and advance it near the cervical canal into the uterine cavity until resistance is felt
 3. Load the embryo(s) into the loading catheter #2000 Semtrac W – C (pink handle) loading cannula, following your standard protocol.
 4. Insert loading catheter #2000 Semtrac W - C (pink handle) into the guiding catheter #2000 Semtrac W-A (white handle).
 5. The markings on the proximal end of the loading cannula #2000 Semtrac W – C (pink handle) indicate the distance the loading cannula has advanced beyond the tip of guiding catheter #2000 Semtrac W – A (white handle).
 6. The distal tip of the catheter should then be at the uterine fundus.
- Note:** If any difficulty with insertion or bending of the catheter occurs, gently rotate the catheter while advancing it close to the cervix. If the uterus is significantly flexed anteriorly or posteriorly, gentle traction on the cervix with a tenaculum may be required to accomplish full insertion. In no case should force be used against resistance to achieve insertion.
7. Slowly release the embryos into the uterus.
 8. Following the complete expulsion of the embryos, slowly withdraw the catheter from the uterus and discard it. Remove the speculum and allow the patient to rest.
 9. Once the set is removed from the patient, inspect the loading catheter #2000 Semtrac W – C (pink handle) to verify embryo release.
 10. Discard after use

Directions for use for the #2000 Semtrac W-180 T/ #2000 Semtrac W-240 T:

1. Place patient in lithotomy or knee/chest position.
2. Insert vaginal speculum to expose cervix.
3. Use malleable/flexible obturator with guide catheter to traverse cervix if necessary. Remove malleable/flexible obturator prior to placement of trial/mock catheter. (May be performed under ultrasound guidance if desired.)
4. Introduce trial/mock catheter through guide sheath.
5. Remove trial/mock catheter from guide sheath.
6. If so desired, move on to embryo transfer using approved catheter and refer to instructions for use for that approved catheter.
7. Upon completion of trial/mock procedure, remove the catheter/catheters, and discard.

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