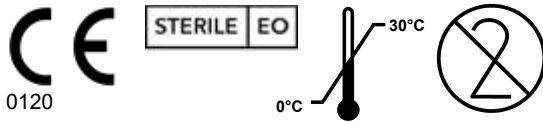
**#2000 SEMTRAC C**

Soft Embryo Transfer Catheter

Important product information, read carefully before use

**Disposable – discard after single use****Re-use may cause sexually transmitted infections****Caution: federal (USA) law restricts this device to sale by or on the order of a physician****Sterile: unless pouch is damaged or opened****INDICATIONS FOR USE:**

Introduction of embryos into the uterine cavity.

**CONTRAINDICATIONS:**

Do not use in the presence of cervical infection.

Do not use in the presence or after recent Pelvic Inflammatory disease.

Do not use for intrafallopian procedures.

**WARNINGS:**

Not sterile if pouch is damaged or opened, discard immediately.

**PRECAUTIONS:**

By virtue of its flexibility and small diameter, the #2000 Semtrac C/ #2000 C TIS can be introduced into the uterus with minimal danger of uterine wall perforation. However, care should be taken, prior to its insertion, to ascertain the depth of the uterus and any uterine anteversion or retroversion present in individual patients. (This is preferably done during a previous examination and not on the day of the artificial insemination when disturbance of the uterus should be minimized).

In no case should the device be forced against digitally felt resistance.

If passage of the #2000 Semtrac C/ #2000 C TIS through the cervical canal is not found to be easily achievable, the device should not be forced and the possibility of the presence of pathologic cervical stenosis considered.

The #2000 Semtrac C/ #2000 C TIS should be used by healthcare professionals only

**ADVERSE REACTIONS:**

No adverse reactions associated with the use of the #2000 Semtrac C/ #2000 C TIS have been reported. However, as with any instrument, which passes through the internal cervical os, mild cramping may be expected.

In every case the direction for use should be followed; taking note of contra-indications, warnings and precautions.

**Manufactured by:**

Gynetics Medical Products N.V.  
Rembert Dodoensstraat 51  
3920 Lommel, Belgium

**HOW SUPPLIED:**

- #2000 Semtrac C
- #2000 C TIS

**PACKAGED:**

3 boxes of 25 individually sterile packed, disposable loading catheters

**DIRECTIONS FOR USE:**

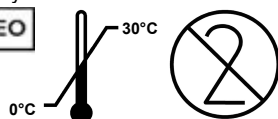
1. With the patient in the dorsal lithotomy position expose the uterine cervix to view with a vaginal speculum.
2. Load the embryo(s) into the loading catheter following your standard protocol.
3. Insert the loading catheter
4. Having previously determined the depth and direction of the patient's uterus, gently advance the catheter into the external cervical os and advance it through the cervical canal into the uterine cavity until resistance is felt. The distal tip of the catheter should then be at the uterine fundus. Now retract the catheter approximately 1.0 cm. Gently rotate the catheter to remove any excess mucus from the tip. **Note:** The tip of the TIS loading catheters can be viewed under ultrasound.
5. Slowly release the embryos into the uterus.
6. Following the complete expulsion of the embryos, slowly withdraw the catheter from the uterus and discard it. Remove the speculum and allow the patient to rest.
7. Once the set is removed from the patient, inspect the loading catheter to verify embryo release.
8. Discard after use

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