
**#4502 A and #4502 B Seminar**  
Intra uterine Insemination Cannula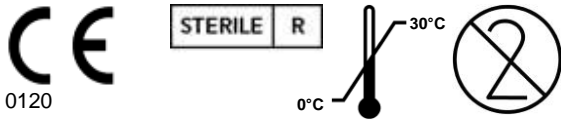

Important product information  
Read carefully before use

**Disposable – discard after single use**  
**Re-use may cause sexually transmitted infections**  
**Caution: federal (USA) law restricts this device to be used by a physician**  
**Sterile: unless pouch is damaged or opened**

**INDICATIONS FOR USE:**

Introduction of washed spermatozoa into the uterine cavity or cervix.

**CONTRAINDICATIONS:**

Do not use in the presence of cervical infection.  
Do not use in the presence or after recent Pelvic Inflammatory disease.

**WARNINGS:**

Always use washed spermatozoa when performing intrauterine (including intracervical) artificial insemination. The introduction of unwashed spermatozoa into the uterus with the #4502A /#4502B or by whatever other means can result in severe adverse reaction which may include anaphylactic shock. Please refer to published medical literature for methods of preparing spermatozoa for intrauterine artificial insemination before performing this procedure.

**PRECAUTIONS:**

By virtue of its flexibility and small diameter, the Seminar Rigid # 4502 A and the Seminar Flexible #4502 B can be introduced into the cervix/uterus with minimal danger of (uterine wall) perforation. However, care should be taken, prior to its insertion, to ascertain the depth of the uterus and any uterine ante flexion or retro flexion present in individual patients. (This is preferably done during a previous examination and not on the day of the artificial insemination when disturbance of the uterus should be minimized).

In no case should the device be forced against digitally felt resistance.

If passage of the Seminar #4502 A/B through the cervical canal is not found to be easily achievable, the device should not be forced and the possibility of the presence of pathologic cervical stenosis considered.

The Seminar should be used by healthcare professionals only

**ADVERSE REACTIONS:**

No adverse reactions associated with the use of the Seminar #4502 A/B have been reported. However, as with any instrument, which passes through the internal cervical os, mild cramping may be expected. In every case the direction for use should be followed; taking note of contra-indications, warnings and precautions.

**Manufactured by:**  
Gynetics Medical Products N.V.  
Rembert Dodoensstraat 51  
3920 Lommel, Belgium

**HOW SUPPLIED:**

- #4502 A Seminar Rigid
- #4502 B Seminar Flexible.


**PACKAGED:**

5 boxes of 25 individually sterile packed, disposable devices.

**DIRECTIONS FOR USE:**

1. With the patient in the dorsal lithotomy position expose the uterine cervix to view with a vaginal speculum.
2. Firmly attach a syringe (preferably 5ml or 3ml) with the plunger fully depressed, to the Seminar at the proximal end.
3. Draw 1.0 ml of air into the syringe. Then place the distal tip of the Seminar into the specimen of WASHED SPERMATOOZOA. Now aspirate the desired quantity of the specimen by retraction of the syringe plunger.
4. Slowly inject the washed spermatozoa into the uterine cavity by depressing the syringe plunger fully. The 1.0 ml of air initially drawn into the syringe prior to loading the cannula with the specimen will help to assure that all the specimen is expelled during the initial injection.
5. Following the complete expulsion of the sperm, withdraw the cannula from the uterus and discard it.
6. Remove the speculum and allow the patient to rest.

**#4502 A and #4502 B Seminar**  
Intra uterine Insemination Cannula

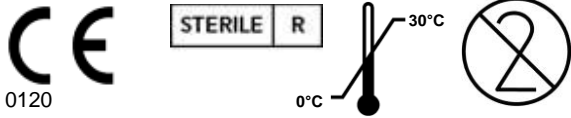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