
#3100, #3125, #3145
Handling Pipette

0120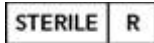

Important product information, read carefully before use

Disposable – discard after single use
Re-use may have a negative influence on the development of the embryo.
Caution: federal (USA) law restricts this device to sale by or on the order of a physician
Sterile: unless pouch is damaged or opened

INDICATIONS FOR USE:

Dish to dish transfer of embryo's and oocytes.

CONTRAINDICATIONS:

There are no known contraindications for the use of this device.

WARNINGS:

Not sterile if pouch is damaged or opened, discard immediately.

PRECAUTIONS:

Assisted reproduction microtools should be comprised of embryo compatible materials.

Chromosomal and congenital abnormalities are complications which may be expected due to the advanced maternal age of the patient population, defects induced through ovulation stimulation, and in vitro manipulation of gametes. When maternal age adjustments are made, the incidence of abnormalities associated with assisted reproduction procedures approximates those of the general population.

The Handling Pipette should be used by healthcare professionals only.

Manufactured by:
Gynetics Medical Products N.V.
Rembert Dodoensstraat 51
3920 Lommel, Belgium

HOW SUPPLIED:

- #3100
- #3100 TP
- #3125
- #3125 TP
- #3145
- #3145 TP

PACKAGED:

- #3100 / # 3125 / # 3145 : 5 boxes of 50 individually sterile packed, disposable devices
- #3100 TP / # 3125 TP / # 3145 TP : 5 boxes of 25 sterile packed pouches, of 10 disposable pipettes.

DIRECTIONS FOR USE:

1. Attach the Handling Pipette to a standard syringe.
2. Aspirate the embryo or oocyte
3. Expel the embryo or oocyte and gently withdraw the Handling pipette from dish.

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