
#2000 SEMTRAC Vision set
Soft Embryo Transfer Catheter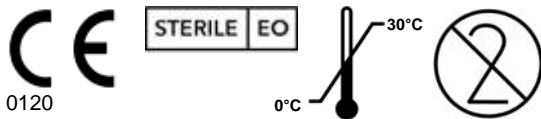

Important product information, read carefully before use

Disposable – discard after single use
Re-use may cause sexually transmitted infections
Caution: federal (USA) law restricts this device to sale by or on the order of a physician
Sterile: unless pouch is damaged or opened

INDICATIONS FOR USE:

Introduction of embryos into the uterine cavity.

CONTRAINDICATIONS:

Do not use in the presence of cervical infection.
Do not use in the presence or after recent Pelvic Inflammatory disease.
Do not use for intrafallopian procedures.

WARNINGS:

Not sterile if pouch is damaged or opened, discard immediately.

PRECAUTIONS:

By virtue of its flexibility and small diameter, the #2000 Semtrac Vision Set can be introduced into the uterus with minimal danger of uterine wall perforation. However, care should be taken, prior to its insertion, to ascertain the depth of the uterus and any uterine anteversion or retroversion present in individual patients. (This is preferably done during a previous examination and not on the day of the artificial insemination when disturbance of the uterus should be minimized).

In no case should the device be forced against digitally felt resistance.

If passage of the #2000 Semtrac Vision Set through the cervical canal is not found to be easily achievable, the device should not be forced and the possibility of the presence of pathologic cervical stenosis considered.

The Semtrac set vision should be used by healthcare professionals only

ADVERSE REACTIONS:

No adverse reactions associated with the use of the #2000 Semtrac Vision Set have been reported.

However, as with any instrument, which passes through the internal cervical os, mild cramping may be expected. In every case the direction for use should be followed; taking note of contra-indications, warnings and precautions.

Manufactured by:

Gynetics Medical Products N.V.
Rembert Dodoensstraat 51
3920 Lommel, Belgium

HOW SUPPLIED:

- #2000 Semtrac Vision set 5
- #2000 Semtrac Vision set 4.2
- #2000 Semtrac Vision C
- #2000 B Set 4.2/ #2000 B Set 5

PACKAGED:

- #2000 Semtrac Vision set 5/ #2000 Semtrac Vision set 4.2: 3 boxes of 25 individually sterile packed, disposable devices and 5 mandrels.
- #2000 Semtrac Vision C: 3 boxes of 25 individually sterile packed, disposable devices
- #2000 B Set 4.2/ #2000 B Set 5: 5 mandrels per 25 #2000 Semtrac Vision set 5/ #2000 Semtrac Vision set 4.2

DIRECTIONS FOR USE:

1. With the patient in the dorsal lithotomy position expose the uterine cervix to view with a vaginal speculum.
2. Load the embryo(s) into the loading catheter Semtrac Vision set C (pink handle) following your standard protocol.
3. Insert loading catheter Semtrac Vision set C (pink handle) into guiding catheter Semtrac Vision set A (white handle).
4. Having previously determined the depth and direction of the patient's uterus, gently advance the catheter into the external cervical os and advance it through the cervical canal into the uterine cavity until resistance is felt. The distal tip of the catheter should then be at the uterine fundus. Now retract the catheter approximately 1.0 cm by following the guides. Gently rotate the catheter to remove any excess mucus from the tip.

Note: If any difficulty with insertion or bending of the catheter occurs, gently rotate the catheter while advancing it close to the cervix. If the uterus is significantly flexed anteriorly or posteriorly, gentle traction on the cervix with a tenaculum may be required to accomplish full insertion. In no case should force be used against resistance to achieve insertion.

5. Slowly release the embryos into the uterus.
6. Following the complete expulsion of the embryos, slowly withdraw the catheter from the uterus and discard it. Remove the speculum and allow the patient to rest.
7. Once the set is removed from the patient, inspect the loading catheter Semtrac Vision set C (pink handle) to verify embryo release.
8. Discard after use

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