**#4000 The Tulip**

Embryo Transfer catheter

0120

Important information, read carefully before use

**Disposable – discard after single use****Re-use may cause sexually transmitted infections****Caution: federal (USA) law restricts this device to sale by or on the order of a physician****Sterile: unless pouch is damaged or opened****Indications for Use:**

Introduction of embryos into the uterine cavity.

**CONTRAINDICATIONS:**

Do not use in the presence of cervical infection.

Do not use in the presence or after recent Pelvic Inflammatory disease.

Do not use for intrafallopian procedures.

**WARNINGS:**

Not sterile if pouch is damaged or opened, discard immediately.

**PRECAUTIONS:**

By virtue of its flexibility and small diameter, #4000 The Tulip / #4000 The Tulip Memo can be introduced into the uterus with minimal danger of uterine wall perforation. However, care should be taken, prior to its insertion, to ascertain the depth of the uterus and any uterine antelexion or retroflexion present in individual patients. (This is preferably done during a previous examination and not on the day of the artificial insemination when disturbance of the uterus should be minimized).

In no case should the device be forced against digitally felt resistance.

If passage of #4000 The Tulip / #4000 The Tulip Memo through the cervical canal is not found to be easily achievable, the device should not be forced and the possibility of the presence of pathologic cervical stenosis considered.

The Tulip should be used by healthcare professionals only

**ADVERSE REACTIONS:**

No adverse reactions associated with the use of #4000 The Tulip / #4000 The Tulip Memo have been reported. However, as with any instrument, which passes through the internal cervical os, mild cramping may be expected. In every case the direction for use should be followed; taking note of contra-indications, warnings and precautions.

**Manufactured by:**

Gynetics Medical Products N.V.  
Rembert Dodoensstraat 51  
3920 Lommel, Belgium

**HOW SUPPLIED:**

#4000 The Tulip  
#4000 The Tulip Memo  
#4000 The Tulip A  
#4000 Tulip S  
#4000 Tulip L  
#4000 Tulip LS  
#4000 Tulip S TIS  
#4000 Tulip L TIS  
#4000 Tulip LS TIS

#4000 S Memo  
#4000 L Memo  
#4000 LS Memo  
#4000 S Memo TIS  
#4000 L Memo TIS  
#4000 LS Memo TIS  
#4000 B  
#4000 Memo B

**PACKAGED:**

- #4000 The Tulip / #4000 Memo / #4000 Tulip S / #4000 Tulip L / #4000 Tulip LS / #4000 Tulip S TIS / #4000 Tulip L TIS / #4000 Tulip LS TIS / #4000 S Memo / #4000 L Memo / #4000 LS Memo / #4000 S Memo TIS / #4000 L Memo TIS / #4000 LS Memo TIS: 3 boxes of 25 individually sterile packed, disposable sets of devices.
- #4000 Memo B: 5 mandrels per 25 #4000 The Tulip Memo
- #4000 B: 5 mandrels per 25 #4000 S Memo / #4000 L Memo / #4000 LS Memo / #4000 S Memo TIS / #4000 L Memo TIS / #4000 LS memo TIS.
- The Tulip A: 3 boxes of 25 individually sterile packed guiding catheters

**DIRECTIONS FOR USE:**

1. With the patient in the dorsal lithotomy position expose the uterine cervix to view with a vaginal speculum.
2. Having previously determined the depth, set positioner on the guiding catheter (white handle).
3. Having previously determined direction of the patient's uterus, put the mandrel, (blue handle), in guiding catheter (white handle) to pre-shape The Tulip A / The Tulip Memo A.
4. Gently advance the catheter into the external cervical os and advance it through the cervical canal near the uterine cavity until resistance is felt.
5. Remove the mandrel (blue handle) when position is reached
6. Load the embryo(s) into the loading catheter (pink handle) following your standard protocol. Insert the loading catheter. The marking on the proximal end of the tube indicates when the distal tip of the guiding catheter is reached.
7. The distal tip of the loading catheter should then be at the uterine fundus. **Note:** The tip of the TIS loading catheters can be viewed under ultrasound.
8. Slowly release the embryos into the uterus.
9. Following the complete expulsion of the embryos, slowly withdraw the catheter from the uterus and discard it. Remove the speculum and allow the patient to rest.
10. Once the set is removed from the patient, inspect The Tulip C / #4000 The Tulip Memo C to verify embryo release.
11. Discard after use

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