
**#2000 SEMTRAC Vision W**  
Soft Embryo Transfer Catheter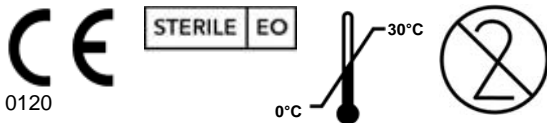

Important information, read carefully before use

**Disposable – discard after single use**

**Sterile: unless pouch is damaged or opened**

**Re-use may cause sexually transmitted infections**

**Caution: federal (USA) law restricts this device to sale by or on the order of a physician**

**INDICATIONS FOR USE:**

Introduction of embryos into the uterine cavity.

**CONTRAINDICATIONS:**

Do not use in the presence of cervical infection.

Do not use in the presence or after recent Pelvic Inflammatory disease.

Do not use for intrafallopian procedures.

**WARNINGS:**

Not sterile if pouch is damaged or opened, discard immediately.

**PRECAUTIONS:**

By virtue of its flexibility and small diameter, the #2000 Semtrac Vision W can be introduced into the uterus with minimal danger of uterine wall perforation. However, care should be taken, prior to its insertion, to ascertain the depth of the uterus and any uterine anteversion or retroversion present in individual patients. (This is preferably done during a previous examination and not on the day of the artificial insemination when disturbance of the uterus should be minimized).

In no case should the device be forced against digital resistance.

If passage of the #2000 Semtrac Vision W through the cervical canal is not found to be easily achievable, the device should not be forced and the possibility of the presence of pathologic cervical stenosis considered.

The #2000 Semtrac Vision W should be used by healthcare professionals only

**ADVERSE REACTIONS:**

No adverse reactions associated with the use of the #2000 Semtrac Vision W have been reported. However, as with any instrument, which passes through the internal cervical os, mild cramping may be expected. In every case the direction for use should be followed; taking note of contra-indications, warnings and precautions.

**Manufactured by:**

Gynetics Medical Products N.V.  
Rembert Dodoensstraat 51  
3920 Lommel, Belgium

**HOW SUPPLIED:**

- #2000 Semtrac Vision W-180
- #2000 Semtrac Vision W-240

**PACKAGED:**

3 boxes of 25 individually sterile packed, disposable sets of devices and 5 mandrels.

**DIRECTIONS FOR USE:**

1. With the patient in the dorsal lithotomy position expose the uterine cervix to view with a vaginal speculum.
  2. Having previously determined the depth and direction of the patient's uterus, gently advance guiding catheter #2000 Semtrac Vision W - A (white handle) cervix cannula into the external cervical os and advance it near the cervical canal into the uterine cavity until resistance is felt
  3. Load the embryo(s) into the loading catheter #2000 Semtrac Vision W - C (pink handle), following your standard protocol.
  4. Insert loading catheter #2000 Semtrac Vision W - C (pink handle) into guiding catheter Semtrac W-A (white handle).
  5. The markings on the proximal end of the loading cannula, loading catheter #2000 Semtrac Vision W - C (pink handle) indicate the distance the loading cannula has advanced beyond the tip of guiding catheter #2000 Semtrac Vision W - A. (white handle)
  6. The distal tip of the catheter should then be at the uterine fundus.
- Note:** If any difficulty with insertion or bending of the catheter occurs, gently rotate the catheter while advancing it close to the cervix. If the uterus is significantly flexed anteriorly or posteriorly, gentle traction on the cervix with a tenaculum may be required to accomplish full insertion. In no case should force be used against resistance to achieve insertion.
7. Slowly release the embryos into the uterus.
  8. Following the complete expulsion of the embryos, slowly withdraw the catheter from the uterus and discard it. Remove the speculum and allow the patient to rest.
  9. Once the set is removed from the patient, inspect the loading catheter #2000 Semtrac Vision W - C (pink handle) to verify embryo release.
  10. Discard after use

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