
#4225 Smooze

Intra uterine Insemination Cannula

0120

Important information, read carefully before use

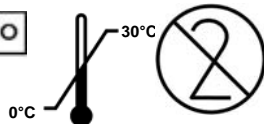**Disposable – discard after single use****Re-use may cause sexually transmitted infections****Caution: federal (USA) law restricts this device to be used by a physician****Sterile: unless pouch is damaged or opened****INDICATIONS FOR USE:**

Intra uterine Artificial Insemination

CONTRAINDICATIONS:

Do not use in the presence of cervical infection

Do not use in the presence or after recent Pelvic Inflammatory disease

**WARNINGS:**

Always use washed spermatozoa when performing intrauterine (including intracervical) artificial insemination. The introduction of unwashed spermatozoa into the uterus with the #4225 Smooze / #4225 Smooze Medium / #4225 Smooze Long/#4225 Long OM or by whatever other means can result in severe adverse reaction which may include anaphylactic shock. Please refer to published medical literature for methods of preparing spermatozoa for intrauterine artificial insemination before performing this procedure.

PRECAUTIONS:

By virtue of its flexibility and small diameter, the #4225 Smooze / #4225 Smooze Medium / #4225 Smooze Long /#4225 Long OM can be introduced into the uterus with minimal danger of uterine wall perforation. However, care should be taken, prior to its insertion, to ascertain the depth of the uterus and any uterine ante flexion or retroflexion present in individual patients. (This is preferably done during a previous examination and not on the day of the artificial insemination when disturbance of the uterus should be minimized). In no case should the device be forced against digitally felt resistance.

If passage of the #4225 Smooze / #4225 Smooze Medium / #4225 Smooze Long/#4225 Long OM through the cervical canal is not found to be easily achievable, the device should not be forced and the possibility of the presence of pathologic cervical stenosis considered.

The #4225 Smooze / #4225 Smooze Medium / #4225 Smooze Long/#4225 Long OM should be used by healthcare professionals only

ADVERSE REACTIONS:

No adverse reactions associated with the use of the #4225 Smooze / #4225 Smooze Medium / #4225 Smooze Long/ #4225 Long OM have been reported. However, as with any instrument, which passes through the internal cervical os, mild cramping may be expected. In every case the direction for use should be followed; taking note of contra-indications, warnings and precautions.

Manufactured by:

Gynetics Medical Products N.V.

Rembert Dodoensstraat 51

3920 Lommel, Belgium

HOW SUPPLIED:

- #4225 Smooze
- #4225 Smooze Medium
- #4225 Smooze Long
- #4225 Long OM

PACKAGED:

- **#4225 Smooze / #4225 Smooze Medium**
5 boxes of 25 individually sterile packed, disposable devices.
- **#4225 Smooze Long / #4225 Long OM**
3 boxes of 25 individually sterile packed, disposable devices.

DIRECTIONS FOR USE:

1. With the patient in the dorsal lithotomy position expose the uterine cervix to view with a vaginal speculum.
 2. Firmly attach a syringe (preferably 5ml or 3ml) with the plunger fully depressed, to the adaptor at the proximal end of the #4225 Smooze / #4225 Smooze Medium / #4225 Smooze Long/#4225 Long OM.
 3. Draw 1.0 ml of air into the syringe. Then place the distal tip of the #4225 Smooze / #4225 Smooze Medium / #4225 Smooze Long/#4225 Long OM into the specimen of WASHED SPERMATOZOA so that the side port is immersed in the specimen. Now aspirate the desired quantity of the specimen into the lumen of the catheter by retraction of the syringe plunger.
 4. Having previously determined the depth and direction of the patient's uterus, gently introduce the distal tip of the #4225 Smooze / #4225 Smooze Medium / #4225 Smooze Long/#4225 Long OM into the external cervical os and advance it through the cervical canal into the uterine cavity until resistance is felt. The distal tip of the catheter should then be at the uterine fundus. Now retract the #4225 Smooze / #4225 Smooze Medium / #4225 Smooze Long/#4225 Long OM approximately 1.0cm.
- Note:** The small OD of the distal segment of the #4225 Smooze / #4225 Smooze Medium / #4225 Smooze Long/#4225 Long OM permits its easy insertion. If any difficulty with insertion or bending of the catheter occurs, gently rotate the catheter while advancing it close to the cervix. If the uterus is significantly flexed anteriorly or posteriorly, gentle traction on the cervix with a tenaculum may be required to accomplish full insertion. In no case should force be used against resistance to achieve insertion.
5. Slowly inject the washed spermatozoa into the uterine cavity by depressing the syringe plunger fully. The 1.0 ml of air initially drawn into the syringe prior to loading the catheter with the specimen plus the air that was contained in the catheter will help to assure that all the specimen is expelled during the initial injection. Should there be any doubt about the complete expulsion of the specimen, remove the syringe from the adaptor; reload the syringe with air; reattach the syringe to the adaptor; then slowly inject the air to expel any remaining specimen into the uterine cavity.
 6. Following the complete expulsion of the specimen, slowly withdraw the #4225 Smooze / #4225 Smooze Medium / #4225 Smooze Long/#4225 Long OM from the uterus and discard it.
 7. Remove the speculum and allow the patient to rest.

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