#4168 AG Curette Flexible

Aspiration curette
 #4168 AG-4 Curette Flex / #4168 AG-6 Curette Flex /
 #4168 AG-8 Curette Flex

0120

Important information, read carefully before use

Disposable – discard after single use**Re-use may cause sexually transmitted infections**

**Caution: federal (USA) law restricts this device to sale by or
 on the order of a physician**

Sterile: unless pouch is damaged or opened

Indications for Use:

Endometrial suction curette

**Contraindications:**

Do not use in the presence of cervical infection.
 Do not use in the presence or after recent Pelvic Inflammatory
 disease.

Warnings:

Not sterile if pouch is damaged or opened, discard immediately.
 When connected to the suction system, do not move the curette
 sideways

Precautions:

The Curette Flexible should be used by healthcare professionals
 only.

Adverse reactions:

No adverse reactions associated with the use of The Curette
 Flexible have been reported. However, as with any instrument,
 which passes through the internal cervical os, mild cramping may
 be expected. In every case the direction for use should be
 followed; taking note of contra-indications, warnings and
 precautions.

**Manufactured by:**

Gynetics Medical Products N.V.
 Rembert Dodoensstraat 51
 3920 Lommel, Belgium

How supplied:

Product number: #4168 AG-4
 #4168 AG-6
 #4168 AG-8

Packaged:

5 boxes of 25 individually sterile packed, disposable devices.

Directions for use:

The Curette Flexible should be used in combination with a suction
 system.

1. Determine the depth of the uterus, by using a Gynetics
 Hysterometer
2. Dilate the cervix if necessary.
3. Introduce the Curette Flexible through the cervical canal into
 the uterus up to the fundus. Connect the curette to the
 suction system.
 Operate by rotating the curette 360° to aspirate throughout
 the uterus cavity accurately.
 Repeat the operation two or three times.
 Withdraw the Curette Flexible from the uterus while
 continuing the rotating movement.
 Dispose of after use.
4. Do not move the curette sideways. Do not go beyond the
 indicated uterus depth.

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Număr produs: #4168 AG-4