
#4502 A and #4502 B Seminor
Intra uterine Insemination Cannula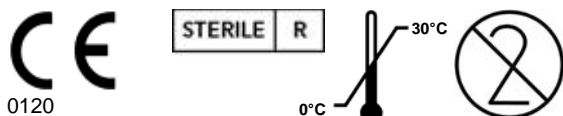

Important product information
Read carefully before use

Disposable – discard after single use
Re-use may cause sexually transmitted infections
Caution: federal (USA) law restricts this device to be used by a physician
Sterile: unless pouch is damaged or opened

INDICATIONS FOR USE:

Introduction of washed spermatozoa into the uterine cavity. Transcervical GIFT, as an introducer and other intrauterine procedures.

CONTRAINDICATIONS:

Do not use in the presence of cervical infection.
Do not use in the presence or after recent Pelvic Inflammatory disease.

WARNINGS:

Always use washed spermatozoa when performing intrauterine (including intracervical) artificial insemination. The introduction of unwashed spermatozoa into the uterus with the IUI SMOOZE or by whatever other means can result in severe adverse reaction which may include anaphylactic shock. Please refer to published medical literature for methods of preparing spermatozoa for intrauterine artificial insemination before performing this procedure.

PRECAUTIONS:

By virtue of its flexibility and small diameter, the Seminor Rigid # 4502 A and the Seminor Flexible #4502 B can be introduced into the cervix/uterus with minimal danger of (uterine wall) perforation. However, care should be taken, prior to its insertion, to ascertain the depth of the uterus and any uterine ante flexion or retro flexion present in individual patients. (This is preferably done during a previous examination and not on the day of the artificial insemination when disturbance of the uterus should be minimized).

In no case should the device be forced against digitally felt resistance.

If passage of the Seminor #4502 A/B through the cervical canal is not found to be easily achievable, the device should not be forced and the possibility of the presence of pathologic cervical stenosis considered.

The Seminor should be used by healthcare professionals only

ADVERSE REACTIONS:

No adverse reactions associated with the use of the Seminor #4502 A/B have been reported. However, as with any instrument, which passes through the internal cervical os, mild cramping may be expected. In every case the direction for use should be followed; taking note of contraindications, warnings and precautions.

The Seminor is developed by Dr. J. Vermeiden of the Amsterdam Free University (VU hospital) IVF laboratory.

Manufactured by:
Gynetics Medical Products N.V.
Rembert Dodoensstraat 51
3920 Lommel, Belgium

HOW SUPPLIED:

- #4502 A Seminor Rigid
- #4502 B Seminor Flexible.


PACKAGED:

5 boxes of 25 individually sterile packed, disposable devices.

DIRECTIONS FOR USE:

1. With the patient in the dorsal lithotomy position expose the uterine cervix to view with a vaginal speculum.
2. Firmly attach a syringe (preferably 5ml or 3ml) with the plunger fully depressed, to the Seminor at the proximal end.
3. Draw 1.0 ml of air into the syringe. Then place the distal tip of the Seminor into the specimen of WASHED SPERMATOZOA. Now aspirate the desired quantity of the specimen by retraction of the syringe plunger.
4. Slowly inject the washed spermatozoa into the uterine cavity by depressing the syringe plunger fully. The 1.0 ml of air initially drawn into the syringe prior to loading the cannula with the specimen will help to assure that all the specimen is expelled during the initial injection.
5. Following the complete expulsion of the sperm, withdraw the cannula from the uterus and discard it.
6. Remove the speculum and allow the patient to rest.

#4502 A and #4502 B Seminar
Intra uterine Insemination Cannula

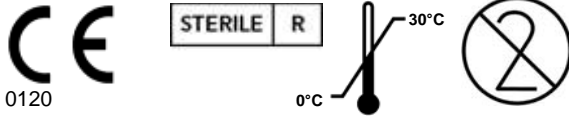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