
**#4219 Emtrac Delphin**  
Embryo Transfer Catheter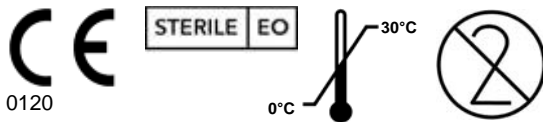

Important information, read carefully before use

**Disposable – discard after single use**  
**Re-use may cause sexually transmitted infections**  
**Caution: federal (USA) law restricts this device to be used by a physician**  
**Sterile: unless pouch is damaged or opened**

**INDICATIONS FOR USE:**

Introduction of embryos into the uterine cavity.

**CONTRAINDICATIONS:**

Do not use in the presence of cervical infection.  
Do not use in the presence or after recent Pelvic Inflammatory disease.  
Do not use for intrafallopian procedures.

**WARNINGS:**

Not sterile if pouch is damaged or opened, discard immediately.

**PRECAUTIONS:**

By virtue of its flexibility and small diameter, the Emtrac Delphin can be introduced into the uterus with minimal danger of uterine wall perforation. However, care should be taken, prior to its insertion, to ascertain the depth of the uterus and any uterine antelexion or retroflexion present in individual patients. (This is preferably done during a previous examination and not on the day of the artificial insemination when disturbance of the uterus should be minimized).

In no case should the device be forced against digitally felt resistance.

If passage of the #4219 Emtrac Delphin through the cervical canal is not found to be easily achievable, the device should not be forced and the possibility of the presence of pathologic cervical stenosis considered.

The Emtrac Delphin should be used by healthcare professionals only.

**ADVERSE REACTIONS:**

No adverse reactions associated with the use of the #4219 Emtrac Delphin have been reported. However, as with any instrument, which passes through the internal cervical os, mild cramping may be expected. In every case the direction for use should be followed; taking note of contra-indications, warnings and precautions.

**Manufactured by:**  
Gynetics Medical Products N.V.  
Rembert Dodoensstraat 51  
3920 Lommel, Belgium

**HOW SUPPLIED:**

- #4219 Emtrac Delphin.
- #4219 Emtrac Delphin TIS

**PACKAGED:**

- **#4219 Emtrac Delphin (TIS)**  
3 boxes of 25 individually sterile packed, disposable devices

**DIRECTIONS FOR USE:**

1. With the patient in the dorsal lithotomy position expose the uterine cervix to view with a vaginal speculum.
  2. Set the positioner as previously measured.
  3. Load syringe with warm 37° medium, connect the syringe to the catheter and flush the catheter by depressing plunger
  4. load catheter by:
    - picking up air,
    - picking up the embryo's in small volume of culture media, note the amount of media collected while picking up the embryos
    - picking up air
    - Picking up the air before and after the embryo can be skipped, if preferred
  5. Gently advance the #4219 Emtrac Delphin into the external cervical os and advance it through the cervical canal into the uterine cavity. **Note:** The tip of the TIS catheter can be viewed under ultrasound.
- Note:** If any difficulty with insertion or bending of the catheter occurs, gently rotate the catheter while advancing it close to the cervix. If the uterus is significantly flexed anteriorly or posteriorly, gentle traction on the cervix with a tenaculum may be required to accomplish full insertion. In no case should force be used against resistance to achieve insertion.
6. With the catheter in position, release the embryos.
  7. Following the complete expulsion of the embryo's, slowly withdraw the catheter from the uterus and discard it. Remove the speculum and allow the patient to rest.
  8. Once the catheter is removed from the patient, inspect the loading catheter to verify embryo release.
  9. Discard after use.

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**Note:** If any difficulty with insertion or bending of the catheter occurs, gently rotate the catheter while advancing it close to the cervix. If the uterus is significantly flexed anteriorly or posteriorly, gentle traction on the cervix with a tenaculum may be required to accomplish full insertion. In no case should force be used against resistance to achieve insertion.

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