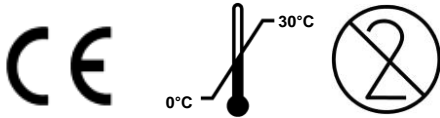
Detecta #4163

Important product information, read carefully before use

**Disposable – discard after single use**

**Re-use may cause sexually transmitted infections**

**Caution: federal (USA) law restricts this device to sale by or on the order of a physician**

#### INDICATIONS AND USAGE

The Detecta is used for cell sampling of the endocervix for cervical screening by cytology.

#### CONTRAINDICATIONS

The Detecta should not be used:

- in pregnancy or suspected pregnancy
- in presence of a suspect, macroscopically visible lesion in the cervix area
- to perform endometrial sampling

In addition to pregnancy, factors adversely affecting the quality of the cell sample include menstruation and vaginal, cervical or pelvic inflammation.

#### WARNINGS

In no case the detecta should be allowed to enter too far into the endocervix. The lower bristles should still be visible at all times.

#### PRECAUTIONS

The Detecta should not be used if the packaging is damaged or the expiration date had passed.

The Detecta should be used by healthcare professionals only

#### ADVERSE REACTIONS

Slight bleeding may occur during sampling.

#### DIRECTIONS FOR USE

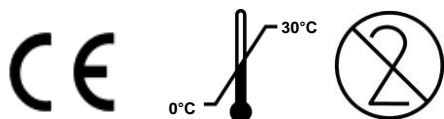
1. gently insert the Detecta into the cervix so that the lower bristles are still visible
2. rotate only once to obtain the sample
3. gently withdraw the Detecta without rotating
4. Roll the sample immediately on a slide
5. fixate the slide immediately
6. sample is ready for examination
7. Discard after use

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