
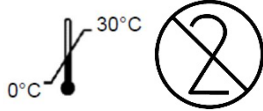
**#4219 EMTRAC SET**

Embryo Transfer catheter

MEDICAL  
DEVICE

1639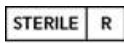

Important information, read carefully before use

**Disposable – discard after single use****Re-use may cause sexually transmitted infections****Caution: federal (USA) law restricts this device to sale by or on the order of a physician****Sterile: unless pouch is damaged or opened****INDICATIONS FOR USE:**

Introduction of embryos into the uterine cavity.

**CONTRAINDICATIONS:**

Do not use in the presence of cervical infection.

Do not use in the presence or after recent Pelvic Inflammatory disease.

Do not use for intrafallopian procedures.

**WARNINGS:**

Not sterile if pouch is damaged or opened, discard immediately.

**PRECAUTIONS:**

By virtue of its flexibility and small diameter, the Emtrac - A can be introduced into the uterus with minimal danger of uterine wall perforation. However, care should be taken, prior to its insertion, to ascertain the depth of the uterus and any uterine antelexion or retroflexion present in individual patients. (This is preferably done during a previous examination and not on the day of the artificial insemination when disturbance of the uterus should be minimized).

In no case should the device be forced against digitally felt resistance. If passage of the Emtrac - A through the cervical canal is not found to be easily achievable, the device should not be forced and the possibility of the presence of pathologic cervical stenosis considered.

The Emtrac Set should be used by healthcare professionals only

**ADVERSE REACTIONS:**

No adverse reactions associated with the use of The #4219 Emtrac Set have been reported. However, as with any instrument, which passes through the internal cervical os, mild cramping may be expected. In every case the direction for use should be followed; taking note of contra-indications, warnings and precautions.

**Manufactured by:**

Gynetics Medical Products N.V.

Rembert Dodoensstraat 51

3920 Lommel, Belgium

**HOW SUPPLIED:**

#4219 Emtrac Set.

**PACKAGED:**

3 boxes of 25 individually sterile packed, disposable devices.

**DIRECTIONS FOR USE:**

1. With the patient in the dorsal lithotomy position expose the uterine cervix to view with a vaginal speculum.
2. Set the positioner as previously measured.
3. Having previously determined the depth and direction of the patient's uterus, gently advance the guiding catheter Emtrac A (white handle) with the Stylet Emtrac B (blue handle) catheter into the external cervical os and advance it through the cervical canal into the uterine cavity until resistance is felt. The distal tip of the catheter should then be at the uterine fundus. Now retract the catheter approximately 1.0 cm by following the guides. Gently rotate the catheter to remove any excess mucus from the tip.

**Note:** If any difficulty with insertion or bending of the catheter occurs, gently rotate the catheter while advancing it close to the cervix. If the uterus is significantly flexed anteriorly or posteriorly, gentle traction on the cervix with a tenaculum may be required to accomplish full insertion. In no case should force be used against resistance to achieve insertion.

4. Remove the Stylet Emtrac B (blue handle) when the position is reached.
5. Load the embryo(s) into the loading catheter Emtrac – C (pink handle) following your standard protocol.
6. With the cannula in position, insert the loading catheter and release the embryos.
7. Following the complete expulsion of the embryos, slowly withdraw the catheter from the uterus and discard it. Remove the speculum and allow the patient to rest.
8. Once the catheter is removed from the patient, inspect the loading catheter to verify embryo release.
9. Discard after use

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