
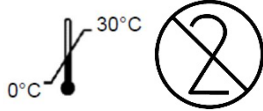
#4551 FAS

Follicle aspiration needle

MEDICAL
DEVICE

1639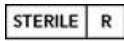

Important product information-Read carefully before use.

Disposable – discard after single use.**Re-use may cause sexually transmitted infections****Caution: federal (USA) law restricts this device to be used by a physician.****Sterile: unless pouch is damaged or opened.****INDICATIONS FOR USE:**

Transvaginal harvesting of oocytes from follicles for IVF and ICSI.

CONTRAINDICATIONS:

Do not use in the presence of (chronic) cervical infection.

Do not use in the presence or after recent Pelvic Inflammatory disease.

Do not use for placement of fertilized oocytes into fallopian tube.

**WARNINGS:**

Please refer to published medical literature before performing this procedure.

PRECAUTIONS:

In no case should the device be forced against digitally felt resistance. If passage of the #4551 Fas is not found to be easily achievable, the device should not be forced.

The Fas should be used by healthcare professionals only

ADVERSE REACTIONS:

No adverse reactions associated with the use of the #4551 Fas have been reported. However, as with any invasive instrument cramping and mild bleeding may be expected. In every case the direction for use should be followed; taking note of contra-indications, warnings and precautions.

**Manufactured by:**Gynetics Medical Products N.V.
Rembert Dodoensstraat 51
3920 Lommel, Belgium**HOW SUPPLIED:**

#4551 FAS

PACKAGED:

2 boxes of 15/25/20 individually sterile packed, disposable devices.

DIRECTIONS FOR USE:

1. With the patient in the dorsal lithotomy position expose the uterine cervix to view with a vaginal speculum
2. Cleanse before inserting the needle following your standard protocol.
3. Connect the needle to the pump
4. Insert the needle following your standard protocol by using ultrasound guidance to locate the follicles.
5. Aspirate (and flush) the oocytes from the follicles and collect the oocytes in a vial (not supplied with #4551 Fas).
6. Withdraw the #4551 Fas from the uterus and discard it.
7. Allow the patient to rest.

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