
#3000 STRIP-ID

Pipette for oocytes manipulation

MEDICAL
DEVICE

1639

STERILE R

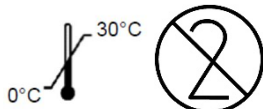Important product information, read carefully before use.

Disposable – discard after single use**Re-use may have a negative influence on the development of the oocyte.****Caution: federal (USA) law restricts this device to sale by or on the order of a physician****Sterile: unless pouch is damaged or opened****INDICATIONS FOR USE:**

Remove cumulus cell layers.

CONTRAINDICATIONS:

There are no known contraindications for the use of this device.

**WARNINGS:**

Not sterile if pouch is damaged or opened, discard immediately.

PRECAUTIONS:

Chromosomal and congenital abnormalities are complications which may be expected due to the advanced maternal age of the patient population, defects induced through ovulation stimulation, and in vitro manipulation of gametes. When maternal age adjustments are made, the incidence of abnormalities associated with assisted reproduction procedures approximates those of the general population.

The pipette should be used by healthcare professionals only.

HOW SUPPLIED:

#3000 Strip-ID125/10; #3000 Strip-ID130/10;
 #3000 Strip-ID135/10; #3000 Strip-ID140/10;
 #3000 Strip-ID145/10; #3000 Strip-ID150/10;
 #3000 Strip-ID170/10; #3000 Strip-ID175/10;
 #3000 Strip-ID200/10; #3000 Strip-ID275/10;
 #3000 Strip-ID300/10; #3000 Strip-ID600/10.

PACKAGED:

- 10 pipettes / individual packaging.
- 15 individual packages in an inner box
- 5 inner boxes/shipping box

DIRECTIONS FOR USE:

1. Attach the pipette to a holder.
2. Aspirate the oocyte in and out the pipette until the cumulus cell layers are removed, the bigger diameters can be used for transfer or handling procedures of the oocyte.
3. Gently withdraw the pipette tip from the dish.
4. Remove the pipette from the holder and discard.

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