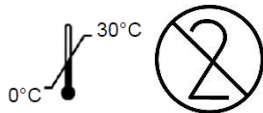
**#4219 EMTRAC C**

Embryo Transfer catheter

MEDICAL  
DEVICE

1639

Important information, read carefully before use

**Disposable – discard after single use****Re-use may cause sexually transmitted infections****Caution: federal (USA) law restricts this device to sale by or on the order of a physician****Sterile: unless pouch is damaged or opened****INDICATIONS FOR USE:**

Introduction of embryos into the uterine cavity.

**CONTRAINDICATIONS:**

Do not use in the presence of cervical infection.

Do not use in the presence or after recent Pelvic Inflammatory disease.

Do not use for intrafallopian procedures.

**WARNINGS:**

Not sterile if pouch is damaged or opened, discard immediately.

**PRECAUTIONS:**

By virtue of its flexibility and small diameter, the Guiding Catheter can be introduced into the uterus with minimal danger of uterine wall perforation. However, care should be taken, prior to its insertion, to ascertain the depth of the uterus and any uterine anteversion or retroflexion present in individual patients. (This is preferably done during a previous examination and not on the day of the artificial insemination when disturbance of the uterus should be minimized).

In no case should the device be forced against digitally felt resistance. If passage of the Guiding Catheter through the cervical canal is not found to be easily achievable, the device should not be forced and the possibility of the presence of pathologic cervical stenosis considered.

**ADVERSE REACTIONS:**

No adverse reactions associated with the use of The #4219 Emtrac C-240 /C-200 have been reported. However, as with any instrument, which passes through the internal cervical os, mild cramping may be expected. In every case the direction for use should be followed; taking note of contra-indications, warnings and precautions.

**Manufactured by:**

Gynetics Medical Products N.V.,  
Rembert Dodoensstraat 51  
3920 Lommel, Belgium

**HOW SUPPLIED:**

- #4219 Emtrac C-200
- #4219 Emtrac C-240

**PACKAGED:**

3 boxes of 25 individually sterile packed, disposable devices.

**DIRECTIONS FOR USE:**

The Emtrac C-240 /C-200 should be used in combination with a reusable guiding catheter.

1. With the patient in the dorsal lithotomy position expose the uterine cervix to view with a vaginal speculum.
2. Load the embryo(s) into the Emtrac - C following your standard protocol.
3. Having previously determined the depth and direction of the patient's uterus, gently advance the guiding catheter into the external cervical os and advance it through the cervical canal into the uterine cavity until resistance is felt. The distal tip of the catheter should then be at the uterine fundus. Now retract the catheter approximately 1.0 cm by following the guides. Gently rotate the catheter to remove any excess mucus from the tip.

**Note:** If any difficulty with insertion or bending of the catheter occurs, gently rotate the catheter while advancing it close to the cervix. If the uterus is significantly flexed anteriorly or posteriorly, gentle traction on the cervix with a tenaculum may be required to accomplish full insertion. In no case should force be used against resistance to achieve insertion.

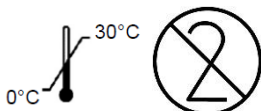
4. Slowly release the embryos into the uterus.
5. Following the complete expulsion of the embryos, slowly withdraw the catheter from the uterus and discard it. Remove the speculum and allow the patient to rest.
6. Once the catheter is removed from the patient, inspect the catheter to verify embryo release.
7. Safely discard after use.

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