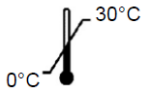
#4163-DETECTA

Cervical Screening Device

MEDICAL
DEVICE

Important product information, read carefully before use

Disposable – discard after single use**Re-use may cause sexually transmitted infections****Caution: federal (USA) law restricts this device to sale by or on the order of a physician****INDICATIONS FOR USE:**

The #4163-Detecta is used for cell sampling of the endocervix for cervical screening by cytology.

CONTRAINDICATIONS:

The #4163 Detecta should not be used:

- in pregnancy or suspected pregnancy
- in presence of a suspect, macroscopically visible lesion in the cervix area
- to perform endometrial sampling

In addition to pregnancy, factors adversely affecting the quality of the cell sample include menstruation and vaginal, cervical or pelvic inflammation.

**WARNINGS:**

In no case the Detecta should be allowed to enter too far into the endocervix. The lower bristles should still be visible at all times.

PRECAUTIONS:

The Detecta should not be used in the packaging is damaged or the expiration date had passed.

The Detecta should be used by healthcare professionals only.

ADVERSE REACTIONS:

No adverse reactions associated with the use of the Detecta have been reported. In every case the direction for use should be followed; taking note of contra-indications, warnings and precautions.

Slight bleeding may occur during sampling.

HOW SUPPLIED:

#4163 Detecta

PACKAGED:

10 units per individual package.
 10 individual packages per box.
 5 boxes per shipping box.

DIRECTIONS FOR USE:

1. With the patient in dorsal lithotomy position expose the uterine cervix to view with a vaginal speculum.
2. Gently insert the Detecta into the cervix so that the lower bristles are still visible.
3. Rotate only once to obtain the sample.
4. Gently withdraw the Detecta without rotating.
5. Roll the sample immediately on a slide.
6. Fixate the slide immediately
7. Sample is ready for examination
8. Discard safely after use

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