REF #4220 IUI / REF #4220 IUI MEMO / REF #4220 PLUS

Intra uterine Insemination Cannula

Important product information

Read carefully before use

Disposable – discard after single use**Re-use may cause sexually transmitted infections****Caution: federal (USA) law restricts this device to be used by a physician****Sterile: unless pouch is damaged or opened****DESCRIPTION:**

The IUI, IUI Plus and IUI Memo are an intrauterine insemination cannula with two lateral distal ports for optimal sperm distribution. The IUI Memo has a Memo function which proactively enables the distal tip to bend according to cervical flexion. The IUI (Memo) is a one piece catheter. The IUI Plus is a two piece catheter which includes additionally a mandrel.

INDICATIONS FOR USE:

Intra uterine Artificial Insemination

CONTRAINDICATIONS:

Do not use in the presence of cervical infection

Do not use in the presence or after recent Pelvic Inflammatory disease

**WARNINGS:**

Always use washed spermatozoa when performing intrauterine (including intracervical) artificial insemination. The introduction of unwashed spermatozoa into the uterus with the #4220 IUI / #4220 IUI Memo / #4220 Plus or by whatever other means can result in severe adverse reaction which may include anaphylactic shock. Please refer to published medical literature for methods of preparing spermatozoa for intrauterine artificial insemination before performing this procedure. In no case should the device be forced against digitally felt resistance. If passage of #4220 IUI / #4220 IUI Memo / #4220 Plus through the cervical canal is not found to be easily achievable, the device should not be forced and the possibility of the presence of pathologic cervical stenosis considered. The IUI should be used by healthcare professionals only.

PRECAUTIONS:

By virtue of its flexibility and small diameter, the #4220 IUI / #4220 IUI Memo / #4220 Plus can be introduced into the uterus with minimal danger of uterine wall perforation. However, care should be taken, prior to its insertion, to ascertain the depth of the uterus and any uterine anteversion or retroversion present in individual patients. (This is preferably done during a previous examination and not on the day of the artificial insemination when disturbance of the uterus should be minimized).

ADVERSE REACTIONS:

No adverse reactions associated with the use of the #4220 IUI / #4220 IUI Memo / #4220 Plus have been reported. However, as with any instrument, which passes through the internal cervical os, mild cramping may be expected. In every case the direction for use should be followed; taking note of contra-indications, warnings and precautions.

Manufactured by:
Gynetics Medical Products N.V.
Rembert Dodoensstraat 51
3920 Lommel, Belgium

HOW SUPPLIED:

- #4220 IUI
- #4220 IUI Memo
- #4220 Plus

PACKAGED:

- #4220 IUI / #4220 IUI Memo: 5 boxes of 25 individually sterile packed, disposable devices.
- #4220 Plus: 3 boxes of 25 individually sterile packed, disposable devices.

DIRECTIONS FOR USE:

1. With the patient in the dorsal lithotomy position expose the uterine cervix to view with a vaginal speculum.
2. Firmly attach a syringe (preferably 5ml or 3ml) with the plunger fully depressed, to the adaptor at the proximal end of the #4220 IUI / #4220 IUI Memo / #4220 Plus.
3. Draw 1.0 ml of air into the syringe. Then place the distal tip of the #4220 IUI / #4220 IUI Memo / #4220 Plus into the specimen of WASHED SPERMATOZOA so that both the lateral ports are immersed in the specimen. Now aspirate the desired quantity of the specimen into the lumen of the catheter by retraction of the syringe plunger.
4. Having previously determined the depth and direction of the patient's uterus, gently introduce the distal tip of the #4220 IUI / #4220 IUI Memo / #4220 Plus into the external cervical os and advance it through the cervical canal into the uterine cavity until resistance is felt. The distal tip of the catheter should then be at the uterine fundus. Now retract the #4220 IUI / #4220 IUI Memo / #4220 Plus approximately 1.0 cm. The positioner may be used to limit insertion of the #4220 IUI / #4220 IUI Memo / #4220 Plus. The default location of the positioner is at 6.0 cm from the external distal tip of the catheter, which corresponds to the ideal position of insemination for the "average" uterine depth of 7.0 cm. Using appropriate aseptic techniques, the positioner should be moved along the outer tube so that the distal end of the positioner is placed at a distance equal to uterine depth minus 1.0 cm from the distal tip of the catheter.

Note: The small OD of the distal segment of the #4220 IUI / #4220 IUI Memo / #4220 Plus and the added control provided by its more rigid proximal segment usually permits its easy insertion. If any difficulty with insertion or bending of the catheter occurs, gently rotate the catheter while advancing it close to the cervix. If the uterus is significantly flexed anteriorly or posteriorly, gentle traction on the cervix with a tenaculum may be required to accomplish full insertion. In no case should force be used against resistance to achieve insertion.

5. With the distal tip of the #4220 IUI / #4220 IUI Memo / #4220 Plus now positioned at 1.0 cm from the uterine fundus, rotate the catheter to bring the marking on the adaptor to the top. This orients the two lateral ports at the distal end of the catheter within the uterine cavity toward the uterine ostia.
6. Slowly inject the washed spermatozoa into the uterine cavity by depressing the syringe plunger fully. The 1.0 ml of air initially drawn into the syringe prior to loading the catheter with the specimen plus the air that was contained in the catheter will help to assure that all the specimen is expelled during the initial injection. Should there be any doubt about the complete expulsion of the specimen, remove the syringe from the adaptor; reload the syringe with air; reattach the syringe to the adaptor; then slowly inject the air to expel any remaining specimen into the uterine cavity.
7. Following the complete expulsion of the specimen, slowly withdraw the #4220 IUI / #4220 IUI Memo / #4220 Plus from the uterus and discard it.
8. Remove the speculum and allow the patient to rest.
9. Discard the catheter in accordance with local regulation.

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