**#3000 STRIP-ID**

Pipette for oocytes manipulation

**HOW SUPPLIED:**

 #3000 Strip-ID125/10     #3000 Strip-ID130/10  
 #3000 Strip-ID135/10     #3000 Strip-ID140/10  
 #3000 Strip-ID145/10     #3000 Strip-ID150/10  
 #3000 Strip-ID170/10     #3000 Strip-ID175/10  
 #3000 Strip-ID200/10     #3000 Strip-ID275/10  
 #3000 Strip-ID300/10     #3000 Strip-ID600/10

Important product information, read carefully before use.

**Disposable – discard after single use**

**Re-use may have a negative influence on the development of the oocyte.**

**Caution: federal (USA) law restricts this device to sale by or on the order of a physician**

**Sterile: unless pouch is damaged or opened**

**DESCRIPTION**

The device is a single-use IVF laboratory micropipette intended for mechanical removal of the cumulus cell layers of the oocyte. Intended patient population: women.

**INDICATIONS FOR USE:**

The device is a micropipette used for oocytes manipulation.

**CONTRAINDICATIONS:**

There are no known contraindications for the use of this device.

**WARNINGS:**

Not sterile if pouch is damaged or opened, discard immediately. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**PRECAUTIONS:**

Chromosomal and congenital abnormalities are complications which may be expected due to the advanced maternal age of the patient population, defects induced through ovulation stimulation, and in vitro manipulation of gametes. When maternal age adjustments are made, the incidence of abnormalities associated with assisted reproduction procedures approximates those of the general population. The device should be used by healthcare professionals only.

1639

**Manufactured by:**  
Gynetics Medical Products N.V.  
Rembert Dodoensstraat 51  
3920 Lommel, Belgium

**PACKAGED:**

- 10 pipettes / individual packaging.
- 15 individual packages in an inner box
- 5 inner boxes/shipping box

**DIRECTIONS FOR USE:**

1. Attach the pipette to a holder.
2. Aspirate the oocyte in and out the pipette until the cumulus cell layers are removed, the bigger diameters can be used for transfer or handling procedures of the oocyte.
3. Gently withdraw the pipette tip from the dish.
4. Remove the pipette from the holder and discard.
5. Discard in accordance with local regulation.

Catalogue number

Batch code

Manufacturer

Temperature limit

Consult instructions for use

CE marking of conformity

Use-by date

Sterilized by irradiation

Do not use if package is damaged

Do not reuse

Medical device

Do not resterilize

Single sterile barrier system with protective packaging inside

Single sterile barrier system with protective packaging

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