#3100 (TP)/#3125 (TP)/#3145 (TP)

Handling Pipette

HOW SUPPLIED:

REF #3100
REF #3100 TP
REF #3125
REF #3125 TP
REF #3145
REF #3145 TP

Important product information, read carefully before use

Disposable – discard after single use

Re-use may have a negative influence on the development of the embryo.

Caution: federal (USA) law restricts this device to sale by or on the order of a physician

Sterile: unless pouch is damaged or opened

DESCRIPTION:

The device is a single-use IVF laboratory micropipette for medical assisted reproduction procedures. The device is used for transferring oocytes and embryos from one dish to another, available in three lengths: 45, 55 and 94 mm. Intended patient population: mature women undergoing medical assisted reproduction procedures.

INDICATIONS FOR USE:

Dish to dish transfer of embryo's and oocytes for medical assisted reproduction procedures.

CONTRAINDICATIONS:

Do not use if the package is opened/damaged; discard immediately.

**WARNINGS:**

The device should be used by healthcare professionals only trained in medical assisted reproduction procedures.

PRECAUTIONS:

Assisted reproduction microtools should be comprised of embryo compatible materials.

Chromosomal and congenital abnormalities are complications which may be expected due to the advanced maternal age of the patient population, defects induced through ovulation stimulation, and in vitro manipulation of gametes. When maternal age adjustments are made, the incidence of abnormalities associated with assisted reproduction procedures approximates those of the general population.

1639

**Manufactured by:**

Gynetics Medical Products N.V.
 Rembert Dodoensstraat 51
 3920 Lommel, Belgium

PACKAGED:

#3100 / #3125 / #3145:

5 boxes of 50 individually sterile packed, disposable devices

#3100 TP / #3125 TP / #3145 TP:

5 boxes of 25 sterile packed 10 disposable pipettes.

DIRECTIONS FOR USE:

1. Attach the device to a standard syringe without putting too much pressure.
2. Aspirate the embryo or oocyte
3. Expel the embryo or oocyte and gently withdraw the device from the dish.
4. Discard the pipette in accordance with local regulation.

Catalogue number

Batch code

Manufacturer

Temperature limit

Consult instructions for use

CE marking of conformity

Use-by date

Sterilized by irradiation

Do not use if package is damaged

Do not reuse

Medical device

Do not resterilize

Single sterile barrier system

Single sterile barrier system with protective packaging

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