#4163-DETECTA

Cervical Screening Device

HOW SUPPLIED:**REF** #4163 Detecta

Important product information, read carefully before use

Disposable – discard after single use**Re-use may cause sexually transmitted infections****Caution: federal (USA) law restricts this device to sale by or on the order of a physician****DESCRIPTION**

The device is used for cell sampling of the endocervix for cervical screening by cytology. Intended patient population: mature women.

INDICATIONS FOR USE:

Cervical Screening Device

CONTRAINDICATIONS:

The device should not be used when one or more of the following conditions exist:

1. Pregnancy or suspicion of pregnancy.
2. Acute pelvic inflammatory disease (PID) or recent treatment for PID.
3. Untreated acute cervicitis, chronic cervicitis, or vaginitis, including bacterial vaginosis, until infection is controlled.

In addition to pregnancy, factors adversely affecting the quality of the cell sample include menstruation and vaginal, cervical or pelvic inflammation.

**WARNINGS:**

In no case the device should be allowed to enter too far into the endocervix. The device should be used by healthcare professionals only. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

PRECAUTIONS:

The device should not be used if the packaging is damaged.

ADVERSE REACTIONS:

No adverse reactions associated with the use of the device have been reported. In every case the direction for use should be followed; taking note of contra-indications, warnings and precautions. Slight bleeding may occur during sampling.

Manufactured by:
Gynetics Medcal Products N.V.,
Rembert Dodoensstraat 51,
3920 Lommel, Belgium

PACKAGED:

10 units per individual package.

5 boxes of 25 individually packed, disposable devices.

DIRECTIONS FOR USE:

1. Check the 'expiry date' of the device.
2. Position the patient in the lithotomy position. Insert vaginal speculum to expose the cervix.
3. Gently insert the device into the cervix.
4. Rotate only once to obtain the sample.
5. Gently withdraw the device without rotating.
6. Roll the sample immediately on a slide.
7. Fixate the slide immediately.
8. Sample is ready for examination.
9. Discard safely after use in accordance with local regulation.

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