#4219 EMTRAC SET

Embryo Transfer catheter

HOW SUPPLIED: #4219 Emtrac Set

Important information, read carefully before use.

Disposable – discard after single use**Re-use may cause sexually transmitted infections****Caution: federal (USA) law restricts this device to sale by or on the order of a physician****Sterile: unless pouch is damaged or opened****DESCRIPTION:**

The device is a three piece embryo transfer catheter with markings to indicate depth, has a handle with luer lock and a positioner for depth adjustment.

INDICATIONS FOR USE:

Introduction of embryos into the uterine cavity.

CONTRAINDICATIONS:

Do not use in the presence of cervical infection, after recent pelvic inflammatory disease, sexually transmitted disease or who is currently pregnant. Do not use if the package is opened/damaged; discard immediately.

**WARNINGS:**

Please refer to published medical literature before performing this procedure. The device should be used by healthcare professionals only. Forcing the catheter may result in damage to the endometrial tissue and bleeding. If passage of the device through the cervical canal is not found to be easily achievable, the device should not be forced and the possibility of the presence of pathologic cervical stenosis considered.

PRECAUTIONS:

By virtue of its flexibility and small diameter, the device can be introduced into the uterus with minimal danger of uterine wall perforation. However, care should be taken, prior to its insertion, to ascertain the depth of the uterus and any uterine ante flexion or retro flexion present in individual patients. (This is preferably done during a previous examination and not on the day of the artificial insemination when disturbance of the uterus should be minimized).

ADVERSE REACTIONS:

No adverse reactions associated with the use of the device have been reported. However, as with any instrument, which passes through the internal cervical os, mild cramping and superficial haemorrhage may be expected. In every case the direction for use should be followed; taking note of contra-indications, warnings and precautions.

Manufactured by:
Gynétics Medical Products N.V.
Rembert Dodoensstraat 51
3920 Lommel, Belgium

PACKAGED:

3 boxes of 25 individually sterile packed, disposable set of devices.

DIRECTIONS FOR USE:

1. Check the expiry date of the device.
2. Position the patient in the lithotomy position. Insert vaginal speculum to expose the cervix. Determine the depth and direction of the patient's uterus.
3. Having previously determined the depth, set positioner on the guiding catheter (white handle).

4. Gently advance the guiding catheter into the external cervical os and advance it through the cervical canal near the uterine cavity until resistance is felt. Gently rotate the catheter to remove any excess mucus from the tip.

Note: If necessary, the use of the mandrel (blue handle) can facilitate insertion of the guiding catheter. This by preforming and reinforcing the guiding catheter.

5. Remove the mandrel (blue handle) when position is reached (if applicable)
6. Attach the syringe to the luer lock of the loading catheter (pink handle). Load the embryo(s) into the loading catheter following your standard protocol. Insert the loading catheter in the guiding catheter.
7. The marking on the proximal end of the tube indicates when the distal tip of the guiding catheter is reached.
8. The distal tip of the loading catheter should then be at the uterine fundus.

Note: If any difficulty with insertion or bending of the catheter occurs, gently rotate the catheter while advancing it close to the cervix. If the uterus is significantly flexed anteriorly or posteriorly, gentle traction on the cervix with a tenaculum may be required to accomplish full insertion. The device should not be used in case of resistance to achieve insertion.

9. Slowly press the plunger of the syringe and expel the embryos into the uterine cavity.
10. Following the complete expulsion of the embryos, slowly withdraw the catheters from the uterus. Inspect the loading catheter (pink handle) to verify embryo release.
11. Remove the speculum and allow the patient to rest.
12. Discard the catheters in accordance with local regulation.

Catalogue number

Batch code

Manufacturer

Temperature limit

Consult instructions for use

CE marking of conformity

Use-by date

Sterilized using irradiation

Do not use if package is damaged

Do not reuse

Medical device

Do not resterilize

Single sterile barrier system

Single sterile barrier system with protective packaging

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