**#4225 SMOOZE**

Intra uterine Insemination Cannula

**HOW SUPPLIED:**

-  #4225 Smooze  
 #4225 Smooze Medium  
 #4225 Smooze Long

Important product information, read carefully before use.

**Disposable – discard after single use**

**Re-use may cause sexually transmitted infections**

**Caution: federal (USA) law restricts this device to be used by a physician.**

**Sterile: unless pouch is damaged or opened**

**DESCRIPTION:**

The device is a flexible intrauterine insemination cannula with a smooth tip with one distal side hole for less traumatic inseminations. It is a one piece catheter available in 3 different lengths. Intended patient population: individuals, male or female with infertility disorder.

**INDICATIONS FOR USE:**

Intra uterine insemination

**CONTRAINDICATIONS:**

Do not use in the presence of cervical infection, after recent pelvic inflammatory disease, sexually transmitted disease or who is currently pregnant. Do not use if the package is opened/damaged; discard immediately.

**WARNINGS:**

Always use washed spermatozoa when performing intrauterine (including intracervical) insemination. The introduction of unwashed spermatozoa into the uterus with the device or by whatever other means can result in severe adverse reaction which may include anaphylactic shock. Please refer to published medical literature for methods of preparing spermatozoa for intrauterine insemination before performing this procedure. If passage of the device through the cervical canal is not found to be easily achievable, the device should not be forced and the possibility of the presence of pathologic cervical stenosis considered. The device should be used by healthcare professionals only. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**PRECAUTIONS:**

By virtue of its flexibility and small diameter, the device can be introduced into the uterus with minimal danger of uterine wall perforation. However, care should be taken, prior to its insertion, to ascertain the depth of the uterus and any uterine anteversion or retroflexion present in individual patients. (This is preferably done during a previous examination and not on the day of the insemination when disturbance of the uterus should be minimized).

**ADVERSE REACTIONS:**

No adverse reactions associated with the use of the device have been reported. However, as with any instrument, which passes through the internal cervical os, mild cramping may be expected. In every case the direction for use should be followed; taking note of contra-indications, warnings and precautions.

1639

**Manufactured by:**  
 Gynetics Medical Products N.V.  
 Rembert Dodoensstraat 51  
 3920 Lommel, Belgium

**PACKAGED:**

- **#4225 Smooze / #4225 Smooze Medium**  
5 boxes of 25 individually sterile packed, disposable devices.
- **#4225 Smooze Long**  
3 boxes of 25 individually sterile packed, disposable devices.

**DIRECTIONS FOR USE:**

1. Check the expiry date of the device.
2. With the patient in the dorsal lithotomy position expose the uterine cervix to view with a vaginal speculum.
3. A) Firmly attach a syringe (preferably 5ml or 3ml) with the plunger fully depressed, to the handle at the proximal end of the device. Draw 1.0 ml of air into the syringe. Then place the distal tip of the device into the specimen of washed spermatozoa so that the side port is immersed in the specimen. Now aspirate the desired quantity of the specimen into the lumen of the catheter by retraction of the syringe plunger.  
 B) Load the syringe according to standard clinical practice. To inseminate attach the device at the tip of the syringe.
4. Determine the depth and direction of the patient's uterus.
5. Insert the catheter. Rotate the catheter to bring the marking on the handle to the top. This orients the two lateral ports at the distal end of the catheter within the uterine cavity toward the uterine ostia.
6. Inject the suspension with the spermatozoa slowly into the cave uteri.
7. Following the complete expulsion of the specimen, slowly withdraw the device from the uterus and discard it in accordance with local regulation
8. Remove the speculum and allow the patient to rest.

Catalogue number

Batch code

Manufacturer

Temperature limit

Consult instructions for use

CE marking of conformity

Use-by date

Sterilization using ethylene oxide

Do not use if package if damaged

Do not reuse

Medical device

Do not resterilize

Single sterile barrier system

Single sterile barrier system with protective packaging

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